

Procedure Summary: Case 000005-04

Procedure: Antebrachial Angular Limb Deformity Correction

Pins Provided: 4x 1.6mm Pins

Printed Materials: 2x Reduction Guides, 2x Osteotomy Guides, 1x Pre-Op Bone, 1x Post-Op Bone

Blade: Stryker SM-3051

Implant: Synthes 2.7mm LCP

Osteotomy


Place osteotomy guide on radius

Pre-drill osteotomy holes to 1.4mm

Install 1.6mm pins

Perform both osteotomies

Cut osteotomy guide and slide off pins

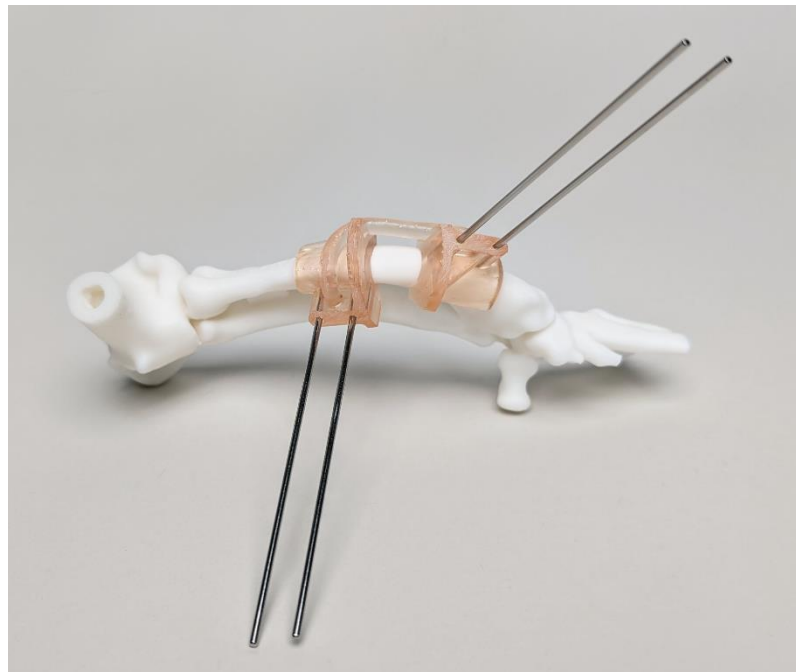

Reduction

Slide reduction guide onto radius using pins

Install plate, cortical, and compression screws

Slide off reduction guide and remove pins

